



**HBV Seroconversion Panel Donor No. 62675  
Part Number: HBV6282**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L)	ABBOTT HBV REAL TIME CP/ML	BIOMERIEUX VIDAS HBsAg	BOEHR. MANNHEIM HBsAg Enzymun-test ES Systems	ORGANON TEKNIKA Hepanostika HBsAg Uni-Form II	ORTHO HBsAg ELISA Testsystem 3	ROCHE Cobas Core HBsAg II EIA
		TEST DATE: 1/27/1998	TEST DATE: UNK	TEST DATE: 4/22/1998	TEST DATE: 4/23/1998	TEST DATE: 4/23/1998	TEST DATE: 3/18/1998	TEST DATE: 4/23/1998
6282-01	10/23/96	7	96	0.11	0.40	0.45	0.01	0.15
6282-02	10/25/96	6	163	0.07	0.38	0.33	0.07	0.19
6282-03	10/30/96	6	553	0.07	0.36	0.36	0.07	0.26
6282-04	11/04/96	7	3257	0.20	0.47	0.42	0.10	0.35
6282-05	11/06/96	10	4929	0.26	0.47	0.49	0.27	0.67
6282-06	11/11/96	7	17896	0.96	0.85	0.97	1.27	1.41
6282-07	11/13/96	8	40064	1.57	1.23	1.33	2.48	2.56
6282-08	11/18/96	15	220168	7.80	5.38	5.96	12.69	11.28
6282-09	11/20/96	5	400640	10.67	7.64	9.03	17.83	15.46
6282-10	11/25/96	9	2151565	37.76	24.91	23.01	51.65	44.63
6282-11	11/27/96	8	4816753	65.34	58.47	MAX.ABS.	MAX.ABS.	MAX.ABS.
6282-12	12/02/96	12	26470027	102.73	MAX.ABS.	MAX.ABS.	MAX.ABS.	MAX.ABS.
6282-13	12/05/96	11	100636235	108.80	MAX.ABS.	MAX.ABS.	MAX.ABS.	MAX.ABS.
6282-14	12/09/96	14	837055740	99.54	MAX.ABS.	MAX.ABS.	MAX.ABS.	MAX.ABS.

PANEL MEMBER	GENETIC SYSTEMS HBsAg (2.0)	ABBOTT AxSYM Core-M	ABBOTT ANTI-HBc TOTAL	ABBOTT ANTI-HBc IgM	ABBOTT HBeAg	ABBOTT PRISM HBsAg	ABBOTT MUREX HBsAg Version 3	BIOTEST ELISA HBsAg
	TEST DATE: 2/4/1998	TEST DATE: 4/7/1998	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 8/27/2009	TEST DATE: 8/27/2009	TEST DATE: 4/6/1998
6282-01	0.117	0.05	0.26	0.094	0.277	1.13	0.52	0.22
6282-02	0.170	0.04	0.24	0.094	0.246	1.08	0.90	0.22
6282-03	0.191	0.06	0.40	0.094	0.238	1.16	0.59	0.22
6282-04	0.202	0.04	0.43	0.075	0.246	1.41	0.68	0.25
6282-05	0.234	0.04	0.52	0.094	0.323	1.66	0.85	0.31
6282-06	0.670	0.04	0.51	0.075	0.292	3.92	1.89	0.63
6282-07	1.170	0.04	0.42	0.075	0.346	6.95	3.28	1.17
6282-08	5.362	0.04	0.26	0.094	1.892	30.14	13.77	5.23
6282-09	6.915	0.04	0.38	0.083	1.523	40.44	18.27	6.90
6282-10	15.777	0.04	0.33	0.075	4.069	32.14	32.51	17.17
6282-11	20.872	0.04	0.36	0.075	5.608	18.49	33.40	24.40
6282-12	25.766	0.04	0.32	0.075	10.885	371.88	33.15	28.82
6282-13	28.277	0.04	0.39	0.075	12.077	400.85	32.63	30.16
6282-14	28.957	0.06	0.53	0.075	14.615	498.56	32.85	29.24

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV6282 Rev03  
Effective Date: 06/23/2021



**HBV Seroconversion Panel Donor No. 62675**  
**Part Number: HBV6282**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	RESEARCH USE PCR PCR (copies/ml)  TEST DATE: 6/2/1998	ABBOTT AxSYM HBsAg Reagent Pack  TEST DATE: 3/19/1998	ABBOTT Auszyme MC Dynamic HBsAg  TEST DATE: 3/31/1998	ABBOTT Auszyme Monoclonal HBsAg  TEST DATE: 4/30/1998	BEHRINGWERKE Enzygnost HBsAg Monoclonal II  TEST DATE: 5/4/1998	SORIN ETI-MAK 3 HBsAg  TEST DATE: 5/5/1998	SORIN (INCSTAR) HBsAg  TEST DATE: UNK
6282-01	500	0.54	0.00	0.07	0.49	0.50	0.235
6282-02	500	0.53	0.10	0.27	0.41	0.17	0.235
6282-03	500	0.56	0.00	0.24	0.39	0.34	0.118
6282-04	7100	0.58	0.07	0.24	0.38	0.50	0.118
9282-05	8200	0.65	0.36	0.20	0.42	0.31	0.157
6282-06	65000	1.10	1.12	0.51	0.85	0.59	0.275
6282-07	19000	1.35	1.71	1.01	1.29	1.75	0.353
6282-08	200000	5.18	11.20	3.84	5.22	8.90	1.686
6282-09	750000	6.03	14.93	4.52	6.57	10.71	2.431
6282-10	2700000	19.05	61.95	29.94	19.49	23.56	6.000
6282-11	6700000	28.95	MAX.ABS.	43.82	33.88	MAX.ABS.	8.080
6282-12	31000000	58.77	MAX.ABS.	MAX.ABS.	31.97	MAX.ABS.	4.750
6282-13	43000000	75.18	MAX.ABS.	MAX.ABS.	MAX.ABS.	MAX.ABS.	4.157
6282-14	>50000000	107.22	MAX.ABS.	MAX.ABS.	31.71	MAX.ABS.	4.666

PANEL MEMBER	SORIN ETI-Core IGMK2  TEST DATE: 5/7/1998	SORIN ETI-AB CoreK2 (Co/S)  TEST DATE: 5/7/1998	ORTHO VITROS ECi HBsAg  TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HBc TOTAL  TEST DATE: 8/30/2018	ORTHO VITROS ECi HBeAg  TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HBe  TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HBc IgM  TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HBs QUANT  TEST DATE: 8/30/2018
6282-01	0.12	0.27	0.06	2.51	0.19	0.22	0.04	0.0
6282-02	0.01	0.27	0.06	2.10	0.32	0.24	0.04	0.0
6282-03	0.02	0.28	0.06	2.57	0.19	0.19	0.03	0.0
6282-04	0.06	0.28	0.09	2.71	0.20	0.18	0.03	0.1
9282-05	0.06	0.24	0.17	2.70	0.24	0.19	0.03	0.1
6282-06	0.06	0.22	1.08	2.63	0.32	0.20	0.03	0.0
6282-07	0.06	0.22	2.50	2.71	0.61	0.18	0.03	0.0
6282-08	0.06	0.28	15.2	2.27	4.09	0.16	0.04	0.2
6282-09	0.02	0.26	22.1	2.60	8.73	0.16	0.03	0.1
6282-10	0.00	0.26	65.5	2.66	64.2	0.07	0.03	0.1
6282-11	0.00	0.27	171	2.47	133	0.05	0.03	0.0
6282-12	0.00	0.26	852	2.37	844	0.01	0.04	0.1
6282-13	0.00	0.25	1820	2.63	1940	0.00	0.03	0.7
6282-14	0.01	0.30	5720	2.05	4260	0.00	0.05	0.2

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV6282 Rev03  
 Effective Date: 06/23/2021