



**HIV Seroconversion Panel Donor No. 64954  
Part Number: HIV9011**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	CHIRON HIV-1 bDNA	ROCHE COBAS PCR Copies/mL	ABBOTT HIV-1 p24 Ag	ABBOTT HIV-1 p24 Ag NT%	COULTER HIV-1 p24 Ag	ABBOTT AxSYM HIV AB ½ EIA	ABBOTT HIV AB ½ EIA rDNA	ABBOTT MUREX HIV Ab ½ + O 3 <sup>RD</sup> Generation
		TEST DATE: 8/20/1998	TEST DATE: UNK	TEST DATE: 8/27/1998	TEST DATE: 9/3/1998	TEST DATE: 8/26/1998	TEST DATE: 9/16/1998	TEST DATE: 7/21/1998	TEST DATE: UNK
9011-01	09/07/97	55	<20	0.31	NEGATIVE	0.56	0.29	0.170	0.423
9011-02	09/11/97	55	<20	0.28	NEGATIVE	0.48	0.31	0.256	0.446
9011-03	09/16/97	410	<20	0.26	NEGATIVE	0.48	0.30	0.256	0.505
9011-04	09/18/97	59	<20	0.27	NEGATIVE	0.56	0.32	0.170	0.443
9011-05	09/23/97	146	<20	0.27	NEGATIVE	0.48	0.30	0.256	0.449
9011-06	09/25/97	133	<20	0.26	NEGATIVE	0.88	0.28	0.170	0.410
9011-07	09/30/97	87	<20	0.20	NEGATIVE	0.72	0.34	0.170	0.357
9011-08	10/02/97	403	<20	0.27	NEGATIVE	0.64	0.31	0.170	0.367
9011-09	10/07/97	8416	3324	0.27	NEGATIVE	1.04	0.33	0.170	0.370
9011-10	10/15/97	194100	55202	2.43	97.30	9.44	1.25	7.008	5.931
9011-11	10/17/97	436000	53577	4.05	106.10	13.44	2.02	12.649	9.475

PANEL MEMBER	ORTHO VITROS ECi ANTI-HIV ½	ABBOTT PRISM ANTI-HIV ½	ORTHO ECi ANTI-HIV	BIOMERIEUX ORGANON TEKNIKA ANTI-HIV-1	BIOMERIEUX ORGANON VIRNOSTIKA HIV UNI-FORM II PLUS 0	BIOTEST ANTI-HIV ½	DADE BEHRING ENZYGNOST ANTI-HIV ½ PLUS	GENETICS SYSTEMS HIV ½ EIA
	TEST DATE: 3/18/2015	TEST DATE: 8/18/1998	TEST DATE: UNK	TEST DATE: 8/26/1998	TEST DATE: 9/2/1998	TEST DATE: 9/24/1998	TEST DATE: 8/31/1998	TEST DATE: 8/25/1998
9011-01	0.43	0.23	0.24	0.58	0.37	0.007	0.02	0.112
9011-02	0.36	0.22	0.24	0.55	0.39	0.005	0.03	0.102
9011-03	0.38	0.19	0.22	0.48	0.38	0.026	0.00	0.105
9011-04	0.38	0.21	0.23	0.52	0.37	0.008	0.02	0.077
9011-05	0.33	0.22	0.22	0.53	0.36	0.000	0.01	0.115
9011-06	0.72	0.23	0.22	0.60	0.35	0.015	0.02	0.093
9011-07	0.36	0.21	0.22	0.58	0.37	0.020	0.01	0.080
9011-08	0.34	0.23	0.21	0.53	0.41	0.025	0.03	0.080
9011-09	0.36	0.22	0.20	0.57	0.38	0.013	0.02	0.089
9011-10	9.93	1.54	13.1	0.57	1.35	0.081	2.14	0.109
9011-11	18.7	2.60	22.0	0.57	1.94	0.548	4.79	0.109

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHIV9011 Rev06  
Effective Date: 10/25/2021



**HIV Seroconversion Panel Donor No. 64954  
Part Number: HIV9011**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO ANTI-HIV ½ ENHANCED ELISA TEST DATE: 9/15/1998	CAMBRIDGE BIOTECH HIV-1 WESTERN BLOT TEST DATE: UNK	ABBOTT PRISM HIV Ag/Ab COMBO 4 <sup>TH</sup> Generation TEST DATE: UNK	ABBOTT MUREX HIV-1 Ab/Ag COMBO 4 <sup>TH</sup> Generation TEST DATE: UNK	ABBOTT HIV Ag/Ab COMBO ARCHITECT 4 <sup>TH</sup> Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV Ag-Ab 5 <sup>th</sup> Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ag 5 <sup>th</sup> Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ab 5 <sup>th</sup> Generation TEST DATE: UNK
9011-01	0.04	NEGATIVE	0.09	0.280	0.09	0.11	0.11	0.07
9011-02	0.05	NEGATIVE	0.09	0.255	0.11	0.04	0.03	0.01
9011-03	0.04	NEGATIVE	0.07	0.265	0.07	0.08	0.06	0.02
9011-04	0.04	NEGATIVE	0.07	0.265	0.23	0.03	0.03	0.02
9011-05	0.05	NEGATIVE	0.08	0.275	0.16	0.10	0.10	0.04
9011-06	0.05	NEGATIVE	0.07	0.275	0.11	0.28	0.28	0.05
9011-07	0.04	NEGATIVE	0.09	0.245	0.11	0.08	0.08	0.03
9011-08	0.04	NEGATIVE	0.08	0.260	0.08	0.11	0.11	0.05
9011-09	0.04	P66(VF)	0.18	0.350	0.21	0.59	0.59	0.06
9011-10	5.47	P66(VF)	4.33	5.860	3.78	17.58	17.58	0.57
9011-11	8.50	P66(VF)	7.18	8.685	5.12	26.30	26.30	1.21

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)