

# ZeptoMetrix®

## HIV Seroconversion Panel Donor No. 65522 Part Number: HIV9014

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	CHIRON HIV-1 bDNA TEST DATE: 8/20/1998	ABBOTT HIV-1 p24 Ag TEST DATE: 8/27/1998	ABBOTT HIV-1 p24 Ag NT% TEST DATE: 9/3/1998	COULTER HIV-1 p24 Ag TEST DATE: 10/30/1998	ABBOTT AxSYM HIV AB ½ EIA TEST DATE: 9/16/1998	ABBOTT HIV AB ½ EIA rDNA TEST DATE: 8/26/1998	ABBOTT MUREX HIV Ab ½ + O 3 <sup>RD</sup> Generation TEST DATE: 8/20/1998
9014-01	11/13/97	25680	0.49	NEGATIVE	1.15	0.36	0.171	0.36
9014-02	11/15/97	5614	0.37	NEGATIVE	0.48	0.36	0.256	0.41
9014-03	11/23/97	281	0.26	NEGATIVE	0.34	1.86	3.932	6.09
9014-04	11/25/97	<50	0.35	NEGATIVE	0.38	1.49	3.162	7.50
9014-05	12/07/97	<50	0.36	NEGATIVE	0.36	4.15	4.530	7.98
9014-06	12/12/97	<50	0.34	NEGATIVE	0.35	5.57	5.300	7.18
9014-07	12/14/97	<50	0.42	NEGATIVE	0.35	6.53	6.325	7.10

PANEL MEMBER	ORTHO VITROS ECi ANTI-HIV ½ TEST DATE: 3/18/2015	ABBOTT PRISM ANTI-HIV ½ TEST DATE: 8/18/1998	BIOMERIEUX ORGANON TEKNIKA ANTI-HIV-1 TEST DATE: UNK	BIOMERIEUX ORGANON VIRNOSTIKA HIV UNI-FORM II PLUS 0 TEST DATE: 9/2/1998	BIO-RAD MULTISPOT HIV 1 RAPID TEST TEST DATE: UNK	BIO-RAD MULTISPOT HIV 2 RAPID TEST TEST DATE: UNK	BIOTEST ANTI-HIV ½ TEST DATE: 9/24/1998
9014-01	0.24	0.28	0.75	0.41	NEGATIVE	NEGATIVE	0.038
9014-02	0.65	0.25	0.62	0.42	NEGATIVE	NEGATIVE	0.046
9014-03	NOT TESTED	2.67	1.45	1.55	NEGATIVE	NEGATIVE	3.010
9014-04	11.3	2.92	1.00	1.53	POSITIVE	NEGATIVE	4.133
9014-05	17.5	20.99	3.00	9.08	POSITIVE	NEGATIVE	5.742
9014-06	20.2	24.76	2.83	9.75	POSITIVE	NEGATIVE	4.225
9014-07	20.2	27.99	2.67	10.19	POSITIVE	NEGATIVE	5.370

PANEL MEMBER	DADE BEHRING ENZYGNOST ANTI-HIV ½ PLUS TEST DATE: 8/31/1998	GENETICS SYSTEMS HIV ½ EIA TEST DATE: 8/25/1998	ORTHO ANTI-HIV ½ ENHANCED ELISA TEST DATE: 9/15/1998	TRINITY BIOTECH UNIGOLD RECOMBIG EN HIV-1 TEST DATE: UNK	ORAQUICK ADVANCE HIV ½ RAPID Ab TEST DATE: UNK	CAMBRIDGE BIOTECH HIV-1 WESTERN BLOT TEST DATE: UNK
9014-01	0.05	0.147	0.04	NEGATIVE	NEGATIVE	P24
9014-02	0.04	0.166	0.20	NEGATIVE	NEGATIVE	P24
9014-03	2.19	0.591	5.52	NEGATIVE	NEGATIVE	INDETERMINATE
9014-04	1.79	0.792	5.70	NEGATIVE	NEGATIVE	P17 P24 P160
9014-05	3.62	2.655	8.89	NEGATIVE	POSITIVE	P17 P24 P66
9014-06	4.09	3.760	9.84	NEGATIVE	POSITIVE	P17 P24 P66
9014-07	4.64	3.508	9.68	NEGATIVE	POSITIVE	P17 P24 P66

### Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHIV9014 Rev06  
Effective Date: 10/25/2021

# ZeptoMetrix®

## HIV Seroconversion Panel Donor No. 65522 Part Number: HIV9014

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ABBOTT PRISM HIV Ag/Ab COMBO 4 <sup>th</sup> Generation  TEST DATE: UNK	ABBOTT MUREX HIV Ag/Ab COMBO 4 <sup>th</sup> Generation  TEST DATE: UNK	ABBOTT HIV Ag/Ab COMBO ARCHITECT 4 <sup>th</sup> Generation  TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV Ag-Ab 5 <sup>th</sup> Generation  TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ag 5 <sup>th</sup> Generation  TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ab 5 <sup>th</sup> Generation  TEST DATE: UNK
9014-01	0.58	0.782	1.48	NOT TESTED	NOT TESTED	NOT TESTED
9014-02	0.30	0.406	0.35	0.41	0.41	0.05
9014-03	1.64	4.365	18.04	NOT TESTED	NOT TESTED	NOT TESTED
9014-04	1.71	4.178	40.72	11.16	0.12	11.16
9014-05	1.79	17.162	54.69	32.77	0.05	32.77
9014-06	2.35	17.898	37.46	13.55	0.05	13.55
9014-07	2.55	18.127	28.65	9.81	0.09	9.81

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHIV9014 Rev06  
Effective Date: 10/25/2021