



**HIV Seroconversion Panel Donor No. 65790
Part Number: HIV9016**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	CHIRON HIV-1 bDNA	ABBOTT HIV-1 p24 Ag	ABBOTT HIV-1 p24 Ag NT%	COULTER HIV-1 p24 Ag	ABBOTT AxSYM HIV AB ½ EIA	ABBOTT HIV AB ½ EIA rDNA	ABBOTT MUREX HIV Ab ½ + O 3 RD Generation	ORTHO VITROS ECi ANTI-HIV 1/2
		TEST DATE: 8/20/1998	TEST DATE: 10/16/1998	TEST DATE: 9/3/1998	TEST DATE: 10/3/1998	TEST DATE: 9/16/1998	TEST DATE: 8/27/1998	TEST DATE: UNK	TEST DATE: 3/18/2015
9016-01	12/11/97	<50	0.301	NEGATIVE	0.05	0.34	0.256	0.293	0.16
9016-02	12/13/97	<50	0.260	NEGATIVE	0.79	0.35	0.171	0.272	0.16
9016-03	12/18/97	<50	0.260	NEGATIVE	0.05	0.35	0.256	0.283	0.16
9016-04	12/20/97	<50	0.288	NEGATIVE	0.05	0.37	0.171	0.283	0.16
9016-05	12/26/97	<50	0.247	NEGATIVE	0.07	0.34	0.171	0.283	0.16
9016-06	12/29/97	<50	0.247	NEGATIVE	0.08	0.34	0.171	0.297	0.16
9016-07	01/03/98	<50	0.260	NEGATIVE	0.05	0.36	0.171	0.318	0.16
9016-08	01/07/98	7473	0.288	NEGATIVE	0.32	0.35	0.171	0.339	0.16
9016-09	01/10/98	69010	0.767	86.60	3.36	0.33	0.256	0.297	0.16
9016-10	01/14/98	286400	3.014	99.00	13.22	0.38	0.171	0.300	0.27

PANEL MEMBER	ORTHO ECi ANTI-HIV ½	ABBOTT PRISM ANTI-HIV ½	ZEPTOMETRIX HIV p24 Ag (pg/mL)	CAMBRIDGE BIOTECH HIV-1 WESTERN BLOT	BIOMERIEUX ORGANON TEKNIKA ANTI-HIV-1	BIOMERIEUX ORGANON VIRNOSTIKA HIV UNI-FORM II PLUS 0	BIOTEST ANTI-HIV ½	DADE BEHRING ENZYGNOST ANTI-HIV ½ PLUS
	TEST DATE: UNK	TEST DATE: 8/18/1998	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 8/26/1998	TEST DATE: 9/2/1998	TEST DATE: 4/22/1998	TEST DATE: 8/31/1998
9016-01	0.06	0.25	<3.0	NEGATIVE	0.50	0.39	0.010	0.04
9016-02	0.04	0.22	<3.0	NEGATIVE	0.53	0.40	0.000	0.02
9016-03	0.06	0.23	<3.0	NEGATIVE	0.48	0.40	0.015	0.03
9016-04	0.06	0.21	<3.0	NEGATIVE	0.47	0.39	0.003	0.03
9016-05	0.06	0.21	<3.0	NEGATIVE	0.47	0.42	0.007	0.02
9016-06	0.06	0.22	<3.0	NEGATIVE	0.52	0.42	0.002	0.02
9016-07	0.06	0.22	<3.0	NEGATIVE	0.50	0.44	0.021	0.02
9016-08	0.06	0.22	<3.0	NEGATIVE	0.50	0.41	0.007	0.14
9016-09	0.06	0.22	<3.0	NEGATIVE	0.52	0.43	0.000	0.03
9016-10	0.06	0.28	19.2	NEGATIVE	0.50	0.40	0.002	0.02

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHIV9016 Rev06
Effective Date: 06/15/2021

ZeptoMetrix®

HIV Seroconversion Panel Donor No. 65790 Part Number: HIV9016

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	GENETICS SYSTEMS HIV ½ EIA TEST DATE: 8/25/1998	ORTHO ANTI-HIV ½ ENHANCED ELISA TEST DATE: 9/15/1998	ABBOTT PRISM HIV Ag/Ab COMBO 4 th Generation TEST DATE: UNK	ABBOTT MUREX HIV-1 Ab/Ag COMBO 4 th Generation TEST DATE: UNK	ABBOTT ARCHITECT HIV Ag/Ab COMBO 4 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV Ag-Ab 5 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ag 5 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ab 5 th Generation TEST DATE: UNK
9016-01	0.102	0.02	0.06	0.330	0.13	0.08	0.08	0.06
9016-02	0.080	0.01	0.06	0.249	0.07	0.07	0.07	0.02
9016-03	0.083	0.00	0.10	0.249	0.11	0.07	0.07	0.02
9016-04	0.083	0.01	0.07	0.279	0.14	0.10	0.10	0.02
9016-05	0.077	0.01	0.07	0.264	0.11	0.12	0.12	0.06
9016-06	0.089	0.02	0.06	0.279	0.15	0.09	0.09	0.06
9016-07	0.096	0.02	0.09	0.315	0.10	0.08	0.08	0.03
9016-08	0.086	0.01	0.20	0.381	0.21	0.67	0.67	0.05
9016-09	0.083	0.02	1.11	1.345	1.41	5.54	5.54	0.04
9016-10	0.099	0.02	5.94	6.340	4.45	34.36	34.36	0.05

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHIV9016 Rev06
Effective Date: 06/15/2021