



**HIV Seroconversion Panel Donor No. 64469
Part Number: HIV9024**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	CHIRON HIV-1 bDNA	ABBOTT HIV-1 p24 Ag	COULTER HIV-1 p24 Ag	ROCHE ELECSYS HIV p24 Ag	ABBOTT AxSYM HIV AB ½ EIA	ABBOTT HIV AB ½ EIA rDNA	ABBOTT HIV AB ½ EIA rDNA 3 rd Generation	ABBOTT MUREX HIV Ab ½ + O 3 rd Generation
		TEST DATE: 10/26/1999	TEST DATE: 5/11/1999	TEST DATE: 6/18/1999	TEST DATE: 6/10/1999	TEST DATE: 7/14/1999	TEST DATE: 5/11/1999	TEST DATE: 6/14/1999	TEST DATE: UNK
9024-01	06/06/97	<50	0.203	0.241	0.50	0.30	0.129	0.22	0.314
9024-02	06/08/97	<50	0.237	0.241	0.49	0.23	0.121	0.20	0.290
9024-03	06/13/97	<50	0.220	0.241	0.47	0.23	0.112	0.22	0.290
9024-04	06/15/97	<50	0.169	0.361	0.47	0.22	0.112	0.20	0.290
9024-05	06/20/97	<50	0.169	0.145	0.49	0.23	0.138	0.19	0.332
9024-06	06/22/97	<50	0.254	0.241	0.47	0.23	0.147	0.21	0.357
9024-07	06/27/97	<50	0.254	0.361	0.43	0.21	0.138	0.19	0.428
9024-08	07/11/97	<50	0.254	0.361	0.39	0.27	0.103	0.21	0.371
9024-09	07/13/97	<50	0.254	0.361	0.44	0.22	0.121	0.25	0.367
9024-10	07/21/97	<50	0.254	0.361	0.37	0.20	0.095	0.21	0.325
9024-11	07/25/97	12840	0.288	0.602	0.38	0.23	0.138	0.18	0.322
9024-12	07/29/97	>500000	6.610	12.410	37.10	0.24	0.138	0.21	0.343

PANEL MEMBER	BIOMERIEUX ORGANON VIRNOSTIKA HIV UNI-FORM II PLUS 0	BIO-RAD NOVAPATH HIV-1 IMMUNOBLOT	BIOTEST ANTI-HIV ½	ABBOTT HIV AB ½ EIA rDNA IMx	ABBOTT MUREX SUDS ANTI-HIV-1	ABBOTT PRISM ANTI-HIV ½	BIOMERIEUX ORGANON VIRNOSTIKA HIV-1 MICROELISA SYSTEM
	TEST DATE: 7/13/1999	TEST DATE: UNK	TEST DATE: 6/9/1999	TEST DATE: 12/7/1999	TEST DATE: 7/21/1999	TEST DATE: 7/15/1999	TEST DATE: 6/18/1999
9024-01	0.45	NEGATIVE	0.02	0.31	0.14	0.28	0.237
9024-02	0.43	NEGATIVE	0.04	0.30	0.15	0.22	0.267
9024-03	0.43	NEGATIVE	0.03	0.30	0.14	0.28	0.208
9024-04	0.47	NEGATIVE	0.04	0.30	0.18	0.24	0.208
9024-05	0.41	NEGATIVE	0.04	0.31	0.15	0.24	0.208
9024-06	1.84	NEGATIVE	0.02	0.31	0.14	0.24	0.237
9024-07	0.48	NEGATIVE	0.02	0.29	0.14	0.34	0.237
9024-08	0.41	NEGATIVE	0.05	0.29	0.14	0.23	0.237
9024-09	0.27	NEGATIVE	0.05	0.33	0.15	0.23	0.237
9024-10	0.38	NEGATIVE	0.04	0.32	0.16	0.26	0.237
9024-11	0.42	NEGATIVE	0.06	1.07	0.15	0.28	0.237
9024-12	0.49	NEGATIVE	0.05	0.36	0.23	0.33	0.237

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHIV9024 Rev05
Effective Date: 06/14/2021



**HIV Seroconversion Panel Donor No. 64469
Part Number: HIV9024**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	BIOTEST ANTI-HIV-1 TETRA ELISA TEST DATE: 6/8/1999	DADE BEHRING ENZYGNOST ANTI-HIV ½ PLUS TEST DATE: 6/8/1999	GENETICS SYSTEMS HIV ½ PEPTIDE EIA TEST DATE: 6/28/1999	GENETICS SYSTEMS rLAV EIA TEST DATE: 11/15/1999	ORTHO ANTI-HIV ½ ENHANCED ELISA TEST DATE: 6/21/1999	ROCHE ENZYMMUM ANTI-HIV-COMBI TEST DATE: 7/14/1999	SANOFI GENSCREEN ANTI-HIV ½ TEST DATE: 6/21/1999	ABBOTT PRISIM HIV Ag/Ab COMBO 4 TH Generation TEST DATE: UNK
9024-01	0.15	0.12	0.053	NOT TESTED	0.04	0.31	0.13	0.08
9024-02	0.18	0.11	0.049	NOT TESTED	0.04	0.33	0.14	0.13
9024-03	0.16	0.16	0.046	NOT TESTED	0.05	0.33	0.12	0.10
9024-04	0.15	0.14	0.046	NOT TESTED	0.05	0.31	0.12	0.12
9024-05	0.17	0.14	0.056	NOT TESTED	0.05	0.29	0.12	0.15
9024-06	0.15	0.14	0.060	NOT TESTED	0.04	0.32	0.09	0.09
9024-07	0.15	0.12	0.056	NOT TESTED	0.04	0.29	0.11	0.10
9024-08	0.17	0.13	0.042	NOT TESTED	0.05	0.29	0.10	0.08
9024-09	0.15	0.12	0.049	NOT TESTED	0.05	0.30	0.11	0.06
9024-10	0.17	0.13	0.039	NOT TESTED	0.06	0.27	0.10	0.13
9024-11	0.16	0.14	0.056	0.065	0.04	0.30	0.14	0.25
9024-12	0.17	0.15	0.056	0.056	0.05	1.61	0.12	25.94

PANEL MEMBER	ABBOTT MUREX HIV-1 Ab/Ag COMBO 4 TH Generation TEST DATE: UNK	ABBOTT ARCHITECT HIV Ag/Ab COMBO 4 TH Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV Ag-Ab 5 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ag 5 th Generation TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ab 5 th Generation TEST DATE: UNK	ORTHO VITROS ECi ANTI-HIV ½ TEST DATE: 8/30/2018
9024-01	0.249	0.08	0.03	0.03	0.01	0.11
9024-02	0.244	0.08	0.04	0.04	0.01	0.11
9024-03	0.234	0.08	0.08	0.08	0.06	0.11
9024-04	0.249	0.10	0.09	0.09	0.01	0.11
9024-05	0.244	0.10	0.05	0.05	0.00	0.12
9024-06	0.249	0.11	0.10	0.10	0.07	0.11
9024-07	0.249	0.08	0.05	0.05	0.01	0.11
9024-08	0.249	0.06	0.10	0.10	0.03	0.10
9024-09	0.274	0.07	0.06	0.06	0.02	0.11
9024-10	0.305	0.07	0.03	0.02	0.00	0.11
9024-11	0.406	0.38	0.84	0.84	0.02	0.11
9024-12	13.868	30.84	94.57	94.57	0.01	0.11

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHIV9024 Rev05
Effective Date: 06/14/2021