



**HIV Seroconversion Panel Donor No. 66048  
Part Number: HIV9031**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	CHIRON	ABBOTT	COULTER	ROCHE	ABBOTT	ABBOTT	ABBOTT	ABBOTT
		HIV-1 bDNA  TEST DATE: 10/22/1999	HIV-1 p24 Ag  TEST DATE: 5/4/1999	HIV-1 p24 Ag  TEST DATE: 6/10/1999	ELECSYS HIV p24Ag  TEST DATE: 7/6/1999	AxSYM HIV AB ½ EIA  TEST DATE: 7/14/1999	HIV AB ½ EIA rDNA  TEST DATE: 5/4/1999	HIV AB ½ EIA rDNA 3 <sup>rd</sup> Generation  TEST DATE: 7/12/1999	MUREX HIV Ab ½ + O 3 <sup>rd</sup> Generation  TEST DATE: UNK
9031-01	09/12/97	<50	0.397	0.342	0.51	0.22	0.081	0.40	0.326
9031-02	10/05/97	<50	0.342	0.266	0.57	0.23	0.090	0.26	0.337
9031-03	11/25/97	<50	0.370	0.342	0.56	NEG	0.090	0.28	0.278
9031-04	12/08/97	<50	0.384	0.253	0.60	0.27	0.099	0.24	0.267
9031-05	12/18/97	67	0.411	0.380	1.00	0.21	0.081	0.28	0.274
9031-06	12/22/97	<50	0.342	0.506	0.52	0.21	0.090	0.31	0.285
9031-07	12/26/97	<50	0.356	0.506	0.48	0.20	0.090	0.28	0.274
9031-08	12/30/97	<50	0.384	0.278	0.50	0.20	0.099	0.27	0.285
9031-09	01/02/98	<50	0.301	0.253	0.50	0.20	0.090	0.28	0.344
9031-10	01/06/98	<50	0.370	0.253	0.47	0.23	0.063	0.34	0.333
9031-11	01/10/98	<50	0.370	0.253	0.43	0.20	0.072	0.24	0.253
9031-12	01/13/98	<50	0.411	0.456	0.47	0.22	0.081	0.27	0.267
9031-13	01/17/98	<50	0.384	0.253	0.53	0.20	0.090	0.28	0.274
9031-14	01/21/98	197	0.370	0.506	0.81	0.44	0.081	0.29	0.292
9031-15	01/24/98	1493	0.397	0.430	0.51	0.22	0.063	0.35	0.274
9031-16	01/28/98	10507	0.548	0.494	0.76	0.19	0.081	0.28	0.302
9031-17	02/05/98	15166	0.356	0.430	0.82	2.36	2.730	3.45	11.413
9031-18	02/12/98	173075	1.534	5.671	4.64	1.47	1.333	1.99	9.792
9031-19	02/16/98	237938	1.658	2.278	5.76	1.99	1.459	2.06	11.198

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHIV9031 Rev05  
Effective Date: 10/26/2021



**HIV Seroconversion Panel Donor No. 66048  
Part Number: HIV9031**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ABBOTT HIV AB ½ EIA rDNA IMx  TEST DATE: 7/19/1999	ORTHO VITROS ECi ANTI-HIV 1/2  TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HIV 1/2  TEST DATE: 7/21/2015	ABBOTT MUREX SUDS ANTI-HIV-1  TEST DATE: 7/21/1999	ABBOTT PRISM ANTI-HIV ½  TEST DATE: 7/15/1999	BIOMERIEUX ORGANON VIRNOSTIKA HIV-1 MICROELISA SYSTEM  TEST DATE: 6/16/1999	BIOMERIEUX ORGANON VIRNOSTIKA HIV UNI-FORM II PLUS 0  TEST DATE: 6/9/1999	BIO-RAD NOVAPATH HIV-1 IMMUNOBLOT  TEST DATE: UNK
9031-01	0.30	0.06	0.12	0.18	0.32	0.231	0.43	NEGATIVE
9031-02	0.30	0.06	0.12	0.17	0.24	0.259	0.47	NEGATIVE
9031-03	0.30	0.07	0.13	0.17	0.23	0.231	0.53	NEGATIVE
9031-04	0.33	0.06	0.14	0.16	0.28	0.202	0.47	NEGATIVE
9031-05	0.29	0.06	0.13	0.19	0.24	0.259	0.45	NEGATIVE
9031-06	0.28	0.06	0.14	0.18	0.24	0.231	0.45	NEGATIVE
9031-07	0.30	0.06	0.12	0.17	0.24	0.202	0.51	NEGATIVE
9031-08	0.28	0.06	0.12	0.21	0.25	0.231	0.54	NEGATIVE
9031-09	0.28	0.03	0.13	0.18	0.35	0.231	0.64	NEGATIVE
9031-10	0.30	0.05	0.12	0.17	0.24	0.231	0.46	NEGATIVE
9031-11	0.28	0.06	0.12	0.16	0.23	0.231	0.45	NEGATIVE
9031-12	0.27	0.06	0.13	0.17	0.25	0.231	0.61	NEGATIVE
9031-13	0.30	0.07	0.13	0.24	0.22	0.259	0.53	NEGATIVE
9031-14	0.39	0.06	0.15	0.24	0.45	0.231	0.55	NEGATIVE
9031-15	0.27	0.06	0.12	0.18	0.27	0.231	0.52	NEGATIVE
9031-16	0.26	0.08	0.14	0.21	0.25	0.231	0.54	NEGATIVE
9031-17	2.54	25.5	30.4	8.70	2.65	0.403	5.81	NEGATIVE
9031-18	1.64	17.2	17.1	10.59	3.18	0.519	4.93	P24
9031-19	1.83	15.7	15.6	10.59	4.73	0.548	4.58	P24

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHIV9031 Rev05  
Effective Date: 10/26/2021



**HIV Seroconversion Panel Donor No. 66048  
Part Number: HIV9031**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	BIOTEST ANTI-HIV ½ TEST DATE: 6/9/1999	BIOTEST ANTI-HIV-1 TETRA ELISA TEST DATE: 6/8/1999	DADE BEHRING ENZYGNOST ANTI-HIV ½ PLUS TEST DATE: 6/8/1999	GENETICS SYSTEMS HIV ½ PEPTIDE EIA TEST DATE: 6/10/1999	GENETICS SYSTEMS rLAV EIA TEST DATE: 11/15/1999	ORTHO ANTI-HIV ½ ENHANCED ELISA TEST DATE: 6/29/1999	ROCHE ENZYMUM ANTI-HIV-1 COMBI TEST DATE: 7/14/1999
9031-01	0.04	0.26	0.13	0.200	0.107	0.06	0.10
9031-02	0.03	0.26	0.13	0.286	0.110	0.06	0.11
9031-03	0.04	0.23	0.15	0.257	0.132	0.06	0.11
9031-04	0.03	0.28	0.14	0.257	0.114	0.05	0.13
9031-05	0.03	0.28	0.15	0.200	0.107	0.04	0.11
9031-06	0.04	0.29	0.13	0.229	0.101	0.04	0.11
9031-07	0.03	0.28	0.11	0.229	0.110	0.04	0.10
9031-08	0.04	0.30	0.11	0.200	0.110	0.05	0.10
9031-09	0.03	0.26	0.13	0.171	0.126	0.06	0.10
9031-10	0.03	0.29	0.13	0.257	0.091	0.06	0.10
9031-11	0.03	0.26	0.14	0.257	0.107	0.06	0.11
9031-12	0.03	0.28	0.13	0.257	0.126	0.04	0.10
9031-13	0.03	0.31	0.14	0.143	0.110	0.05	0.10
9031-14	0.02	0.25	0.16	0.286	0.139	0.06	0.10
9031-15	0.01	0.27	0.13	0.229	0.095	0.05	0.11
9031-16	0.03	0.27	0.14	0.257	0.095	0.07	0.12
9031-17	3.09	7.57	3.37	0.371	0.464	12.198	4.41
9031-18	6.44	3.85	2.63	3.914	0.880	8.67	19.17
9031-19	6.85	3.96	3.34	3.829	1.237	7.71	18.57

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)



**HIV Seroconversion Panel Donor No. 66048  
Part Number: HIV9031**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	SANOFI GENSCREEN ANTI-HIV ½  TEST DATE: 6/21/1999	ABBOTT PRISM HIV Ag/Ab COMBO 4 <sup>TH</sup> Generation  TEST DATE: UNK	ABBOTT MUREX HIV-1 Ab/Ag COMBO 4 <sup>TH</sup> Generation  TEST DATE: UNK	ABBOTT ARCHITECT HIV Ag/Ab COMBO 4 <sup>TH</sup> Generation  TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV Ag-Ab 5 <sup>th</sup> Generation  TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ag 5 <sup>th</sup> Generation  TEST DATE: UNK	BIO-RAD BioPlex 2200 HIV-1 Ab 5 <sup>th</sup> Generation  TEST DATE: UNK
9031-01	0.06	0.05	0.429	0.07	0.08	0.08	0.04
9031-02	0.05	0.06	0.414	0.07	0.07	0.07	0.03
9031-03	0.05	0.06	0.470	0.09	0.06	0.03	0.06
9031-04	0.04	0.08	0.369	0.12	0.08	0.08	0.08
9031-05	0.05	0.08	0.369	0.10	0.12	0.11	0.05
9031-06	0.06	0.06	0.369	0.16	0.10	0.10	0.03
9031-07	0.06	0.09	0.394	0.10	0.08	0.08	0.02
9031-08	0.05	0.07	0.399	0.08	0.09	0.09	0.04
9031-09	0.06	0.08	0.439	0.08	0.08	0.08	0.06
9031-10	0.06	0.09	0.429	0.11	0.12	0.12	0.04
9031-11	0.06	0.07	1.596	0.08	0.06	0.06	0.04
9031-12	0.04	0.08	0.369	0.08	0.06	0.05	0.06
9031-13	0.05	0.08	0.338	0.11	0.09	0.09	0.03
9031-14	0.08	0.07	0.338	0.06	0.10	0.09	0.10
9031-15	0.06	0.08	0.384	0.11	0.15	0.15	0.00
9031-16	0.06	0.23	0.545	0.28	0.53	0.53	0.05
9031-17	MAX ABS	1.83	15.551	2.24	3.12	0.86	3.12
9031-18	MAX ABS	4.41	11.864	6.10	12.05	6.81	12.05
9031-19	MAX ABS	5.14	14.576	7.77	12.87	9.16	12.87

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)